



Cinta Matriz Metálica

Cinta matriz de acero inoxidable de 0,05mm

Es Cintas matrices TDV para restauraciones de Clase II. Uso: Instalar la cinta matriz en el portamatriz. Insertar el conjunto de cinta matriz y portamatriz. Ajustar el conjunto al diente. Insertar la cuña anatómica en la proximal y realizar el procedimiento restaurador. Después de terminada la restauración, remover la cuña y el conjunto. Realizar el acabamiento y pulimento. Advertencias, precauciones y restricciones: Uso profesional. Uso único. Su reutilización puede causar la contaminación cruzada. Para evitar la deglución o aspiración accidental de este producto se recomienda el aislamiento absoluto del campo operatorio. No utilizar en pacientes que tengan antecedentes de alergia al componente de este producto. Principio físico de funcionamiento: La cinta actúa como una barrera física para el material restaurador. Composición: Acero inoxidable. Versión 04 - 12/2015

 TDV Dental Ltda.
CNPJ 81.591.786/0001-60
XV de Novembro, 9944 | CEP 89107-000
Pomerode/SC - Brasil
Tecn. Resp.: Tais Cugola C. Pereira
CRF/SC 13884
Reg. ANVISA 10291220057
 55 47 3395 - 6115
info@tdv.com.br | www.tdv.com.br

EC REP

Obelis S.A
Bd. Général Wahis 53
1030 Brussels, Belgium
Tel: +(32) 2.732.59.54
E-mail: mail@obelis.net





Metallic Matrix Strip

Stainless steel matrix strip of 0,05mm

En The Metallic Matrix Strips are indicated to be used with photopolymerizable resins in Class II restorations. Use: Install the matrix strip on the matrix holder. Insert the matrix strip and matrix holder set into the interproximal space. Adjust the set to the tooth. Insert the anatomic wedge into the proximal gap and realize the restorative procedure. After concluding the restoration, remove the wedge and the set. Perform the finishing and polishing. Warnings, precautions and restrictions: Professional use only. One use only. Its reuse can cause cross contamination. To avoid and accidental swallowing of this product it's necessary the application of a total isolation technique of the operator field. When there is proven allergy to any of its components, the product must not be used. Physical principle of operation: The strip acts as a physical barrier to the restorative material. Composition: Stainless steel. Version 04 - 10/2014

TDV Dental Ltda
CNPJ 81.591.7867/0001-60
Caixa Postal 200 | CEP 89107-000
Pomerode/SC - Brasil
Téc. Resp. Dr. Elson J. Costa
CRO/SC 1202
Reg. ANVISA 10291220047
☎ 55 47 3395 6115
info@tdv.com.br | www.tdv.com.br

EC REP

Obelis S.A
Bd. Général Wahis 53
1030 Brussels, Belgium
Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03
E-mail: mail@obelis.net

